

# BREEZE<sup>®</sup> 2 Meter Quick Test Guide

## The BREEZE<sup>®</sup> 2 Blood Glucose Monitoring System from Bayer SIMPLE TESTING


### Testing your blood sugar

1

Once you have loaded the 10-test disc (see reverse side), hold the meter in your hand or on a flat surface. Pull **(1a)** and push **(1b)** the meter handle to expose a test strip. The meter will automatically turn on, and the “apply blood” prompt will appear on the screen. The meter is now ready to test.



1a

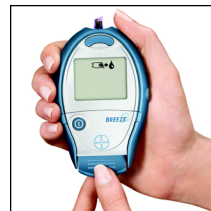

1b

2

Touch the tip of the test strip to the blood drop. Hold in place **(2a)** until the meter beeps and count-down begins from 5 to 1. The result is displayed **(2b)** after 5 seconds and is stored in the meter automatically for future reference.



2a



2b

3

To discard the test strip, hold the meter pointing down over a waste container and press the blue release button at the top. Then turn the meter off.



3



For more information on Bayer products and services visit [www.BayerDiabetes.com](http://www.BayerDiabetes.com).

**NO CODING** ✓

**EASY ACCURACY. ANSWERED BY BAYER.**

0392247

# BREEZE<sup>®</sup> 2 Meter Quick Test Guide

## The BREEZE<sup>®</sup> 2 Meter from Bayer SIMPLE TESTING


### Loading the BREEZE<sup>®</sup> 2 Test Disc

**1** To open the meter, hold it with the screen facing down. Press in on the gray latch on the top edge of the meter, and then pull up.



1


**2** Insert a new 10-test disc, bumpy side up, keeping the meter flat. Gently, snap the meter shut.

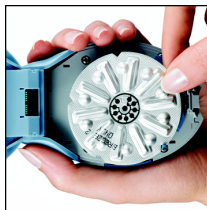

2

**Note:** When you push the “On/Off” button, the screen will display how many test strips remain in the disc in the top left-hand corner.

#### Function Buttons

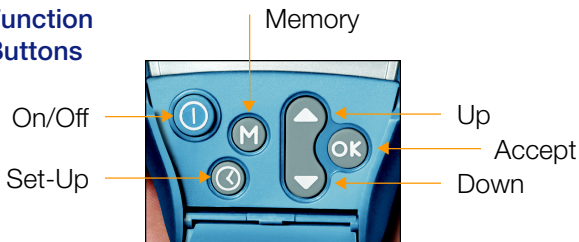

If you have any questions, please refer to the **User Guide for the BREEZE<sup>®</sup> 2 Meter.**

Bayer Customer Service provides personal assistance when you need it. **Call 1-800-348-8100.**

**Note:** New BREEZE 2 Test Strips do not work with the original BREEZE Meter; use only your new BREEZE 2 Meter.



*The BREEZE<sup>®</sup> Meter from Bayer is the first blood glucose meter awarded the **Ease-of-Use Commendation from the Arthritis Foundation.***

