
Your CONTOUR TS Meter:

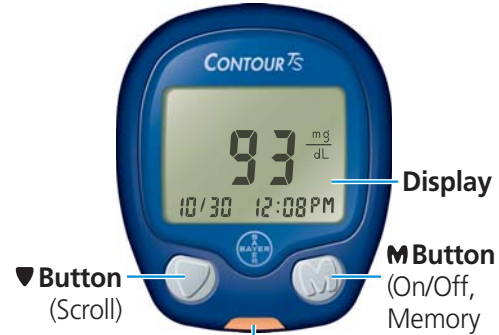
▼ Button (Scroll)

Orange Test Strip Port

Your Contour TS Test Strip:

Sample Tip: The end of the test strip where the blood is drawn in.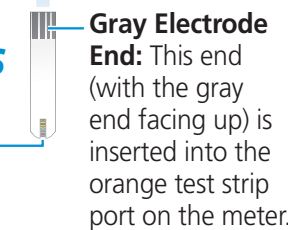

Gray Electrode End: This end (with the gray end facing up) is inserted into the orange test strip port on the meter.

Need Help? Call toll free
1-800-348-8100
www.bayerdiabetes.com/us

Inserting the Test Strip:

Remove a test strip from the bottle and **firmly snap the lid closed. Never store test strips outside the bottle!**

Hold the test strip with the gray end facing up and insert it into the orange test strip port. *No coding required!*

A test strip with a flashing blood drop will appear letting you know the meter is ready to test.

Getting the Blood Drop:

Press the lancing device firmly against the puncture site and press the release button.


Test immediately after you have formed a blood drop. Only 0.6 μ L is needed for a test.


Suggested Actual Drop Size

Testing Your Blood:

Hold the meter with the orange test strip port pointing down or toward you. Immediately touch the **tip** of the test strip to the drop of blood. The blood is drawn into the test strip through the tip.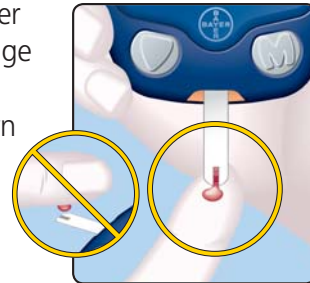

Hold the tip of the test strip in the blood drop until the meter beeps and the 8 second countdown begins. Make sure the orange test strip port is pointing down.
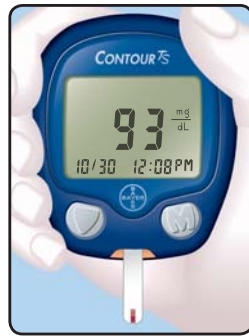
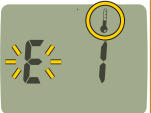
Your test result is displayed and automatically stored in the meter memory.

To turn your meter off, simply remove the test strip. Carefully dispose of the used test strip.

Error Codes and Symbols:

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
	Low Battery.	Replace the battery as soon as possible.
	Temperature out-of-range.	Move to an area that is within the operating range for the meter: 41°F–113°F. Let the meter and test strips adjust to this temperature for twenty minutes before testing.

Refer to your User Guide for complete troubleshooting and error information, or call Bayer Diabetes Care Customer Service: **1-800-348-8100 (24 hours/day)**

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
	The test strip has not filled enough to give an accurate result.	Remove the test strip and repeat the test using a new test strip.
	The meter is sensing a used test strip.	Remove the test strip and retest using a new test strip. Wait until you see the flashing blood drop in the display before adding the blood sample.

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
	Test strip not inserted correctly.	Remove the test strip and insert it properly.
E5 E6 E8 E9	E10 E11 E12 E13	Potential software or hardware issue. Remove the test strip and retest using a new test strip. If you continue to have problems, contact Customer Service.
	Wrong test strip.	Remove the test strip and retest using a CONTOUR TS Test Strip.

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
	Test result is "HI" (above 600 mg/dL)	<ul style="list-style-type: none"> Wash your hands and the test site. Repeat the test using a new test strip. If your result is still "HI" or "LO," contact your physician or healthcare professional immediately.
	Test result is "LO" (below 10 mg/dL).	
CAUTION: Glucose levels above 250 mg/dL or below 50 mg/dL may indicate a potentially serious medical condition.		

Bayer

©2007 Bayer
50184363 Rev. 5/07

 **CONTOUR^{TS}**
Blood Glucose Monitoring System

Quick Reference Guide

Only Use With
CONTOUR[®] TS Blood Glucose Meter
and CONTOUR[™] TS Test Strips
(Read your User Guide and
Lancing Device insert for
complete instructions before
performing a blood glucose test.)

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