
TrueTrack smart system®

For Accurate, Affordable Blood Glucose Monitoring

Meter, Test Strips & System Components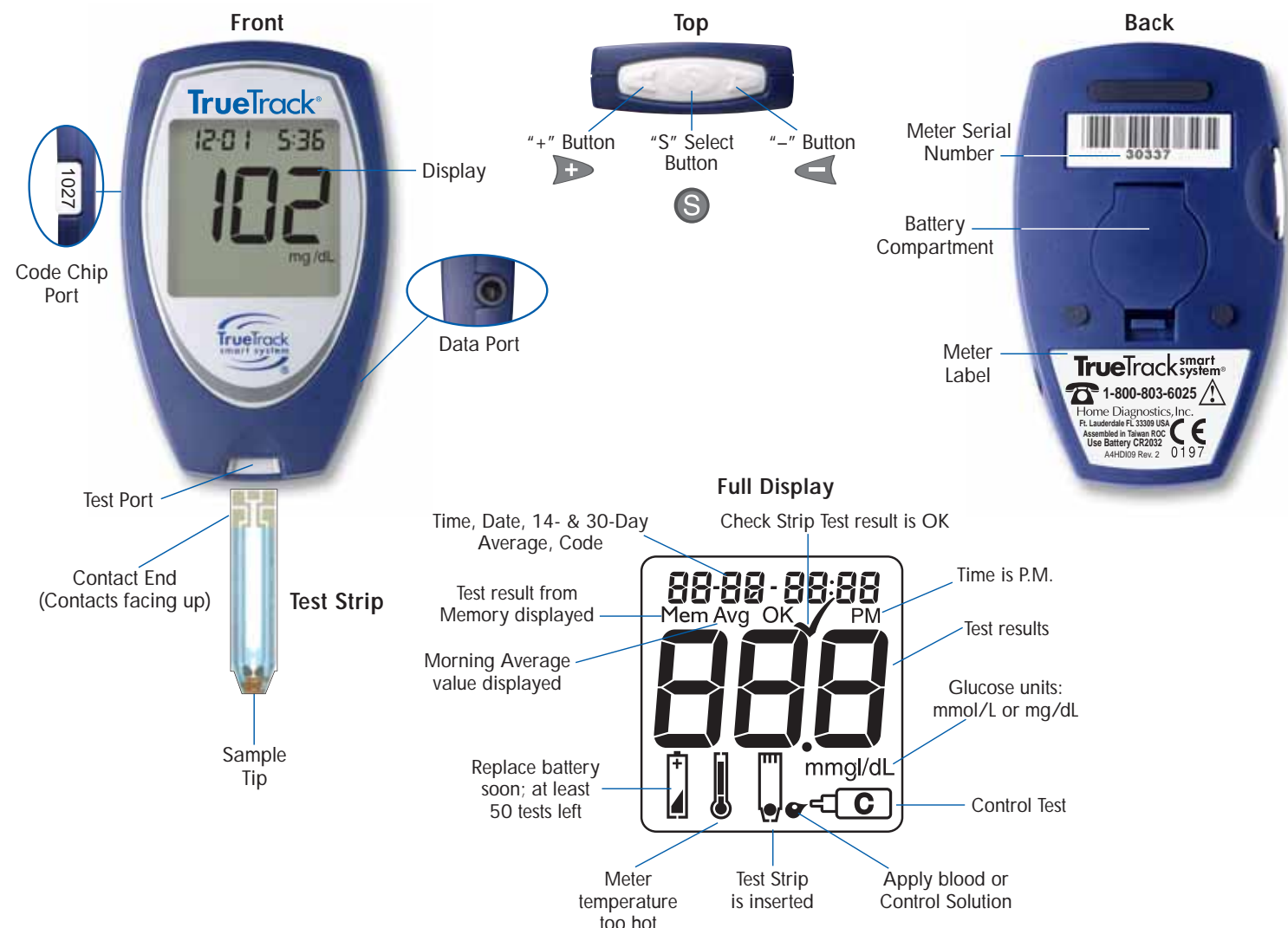

Set Up

Start with the Meter off. Do not insert a Test Strip.

1. Hold down the **S** button until Meter beeps and full Display appears. Meter is now in Set Up mode.

Setting Time:

2. The number for the hour flashes. Press the **+** button to increase or the **-** button to decrease. Press the **S** button when the correct hour is shown.
3. Repeat Step 2 for minutes.

Setting Date:

4. The number for the month flashes. Press the **+** button to increase or the **-** button to decrease. Press the **S** button when the correct month is shown.
5. Repeat Step 4 for day.
6. Repeat Step 4 for year.

Setting Units:


7. The unit of measure (mg/dL or mmol/L) flashes. Press the **+** or **-** button to scroll between. Press the **S** button to select the desired units.
- Press and hold the **S** button until Display goes blank (5 seconds) to save settings and turn Meter off.

⚠ For more detailed information, including a troubleshooting guide, refer to the Owner's Booklet.

Coding

1. Remove Code Chip from box of Test Strips. Match the number printed on Code Chip to Code printed on vial label.
2. With Meter off, insert Code Chip (numbers facing up) into Code Chip Port as shown, locking into place.
3. Insert Test Strip into Test Port to turn Meter on. Code Number appears in Display. Match to Code on vial label and begin testing.

Note: Remove and discard Code Chip in Meter when all vials in Test Strip box are empty.

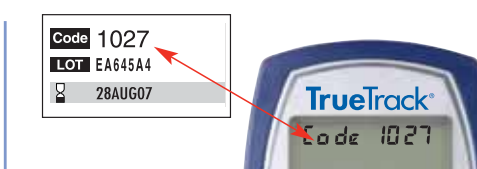Note: Do not use Test Strips if Codes do not match. If "Code ---" shows in Display or Meter does not turn on, see Owner's Booklet. Call Technical Service for assistance at 1-800-803-6025.

Blood Glucose Testing

1. Wash hands with warm, soapy water.
2. Remove Test Strip from vial and recap vial immediately.
3. With Meter off, insert Test Strip (Contacts facing up) into Test Port. Meter turns on.
4. Confirm Meter is properly coded (see Coding above).

5. Lance finger. Let a blood drop form.
 6. Place Sample Tip of Test Strip against blood drop and hold (see illustration). Blood is drawn into Test Strip until Meter beeps. (See Note below).
 7. After countdown, glucose result is displayed with date/time. Remove Test Strip and discard. Meter turns off. Test result is stored in Meter's Memory.
- Note:** Discard Test Strip if Meter did not beep within 5 seconds after placing Tip of Test Strip against blood drop. Repeat test with new Strip and new blood sample. If problem persists, see Owner's Booklet.

Data Management

Morning Averages (14- & 30-day)

To view 14- & 30-day average values, start with Meter off. Press and release **S** button. Display shows 14-day average value and then 30-day average value.

Note: To view Morning Average values, at least one blood glucose test must be performed between 4:00 a.m. – 9:59 a.m., according to Meter clock. Meter clock must be set for actual time to calculate accurate Morning Averages.

Meter switches between 14- & 30-day average values for two minutes before turning off. If there are no Morning Average values, three dashes are displayed.

Viewing Test Results In Memory

To view Memory, press and release the **S** button after Morning Average is displayed (see Morning Averages section). Meter displays most recent test result in Memory with date and time. To scroll forward by date and

time, press the **+** button. To scroll backward by date and time, press the **-** button. Memory stores 365 test results, which are displayed from most recent to oldest.

Always follow the treatment advice given to you by your doctor or diabetes healthcare professional.

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Home Diagnostics, Inc.