

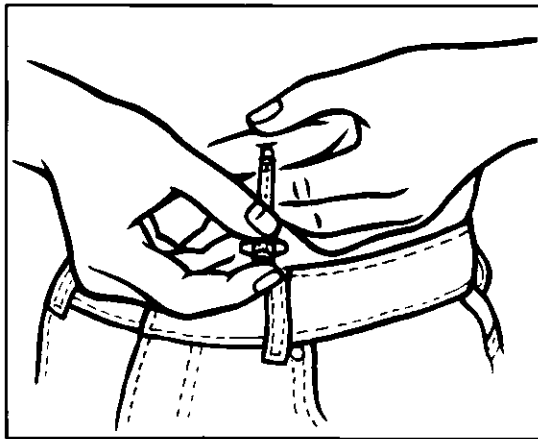
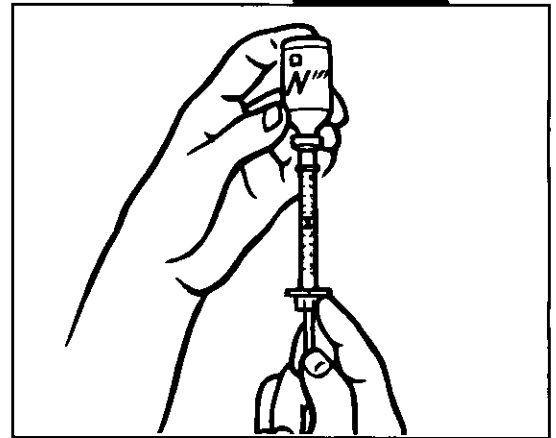
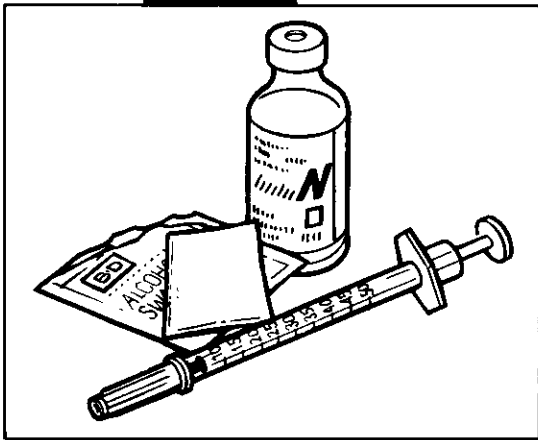
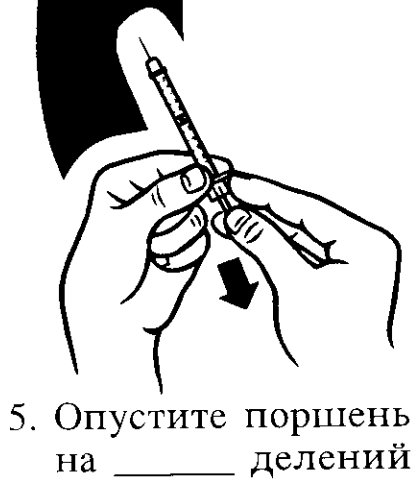
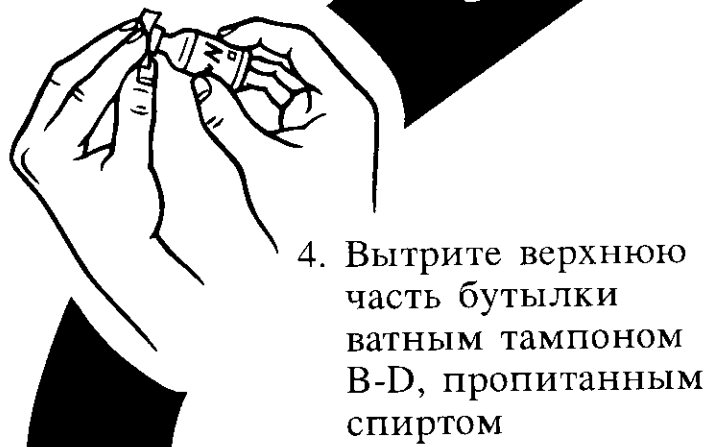
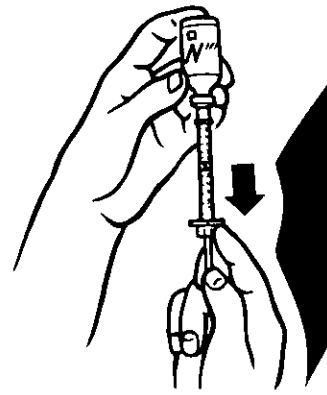



B-D

GETTING STARTED®


Приготовление и самоинъекции инсулина

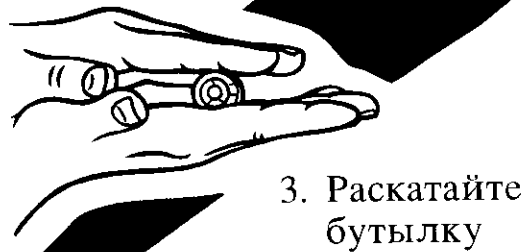

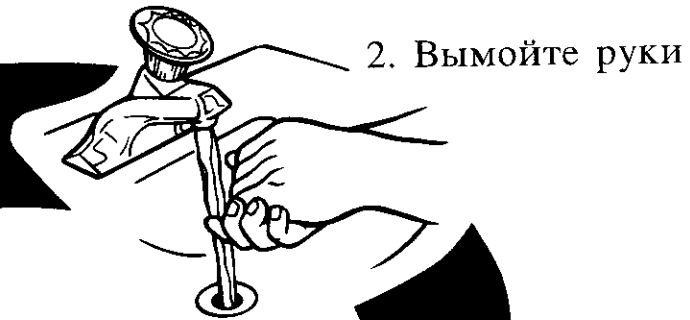

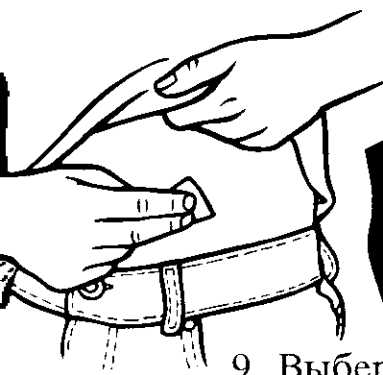


9. Выберите место для инъекции
Протрите его ватным тампоном В-Д, пропитанным спиртом

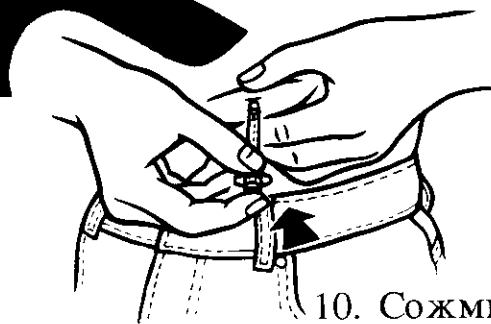


10. Сожмите кожу
Введите иглу в кожу
Введите шприц



8. Опустите поршень на _____ делений

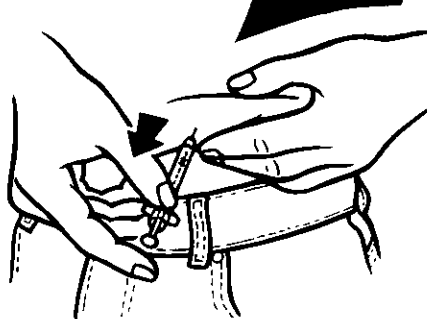

11. Выведите иглу

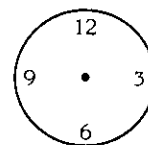

12. Избавляйтесь от шприцев безопасным образом
Проверьте правила своего города

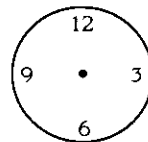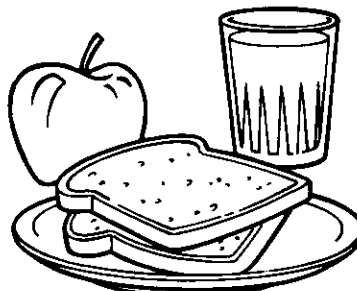


13. Примите пищу через _____ минут.

Дозировка инсулина для:




Имя пациента _____ Сегодняшняя дата: _____

Имя медсестры _____ Номер телефона: _____

Размер шприца: 3/10 cc 1/2 cc 1 cc

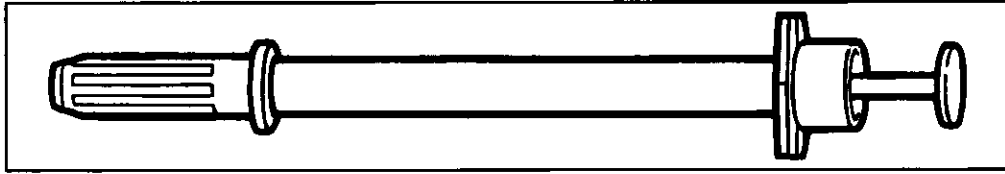


Я принимаю
инсулин: _____
(Фирменное название)

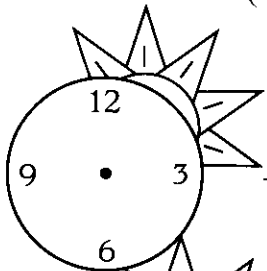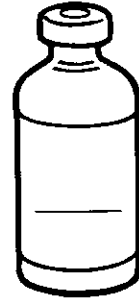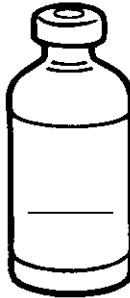

_____ часов утра, перед завтраком

_____ количество единиц


_____ количество единиц

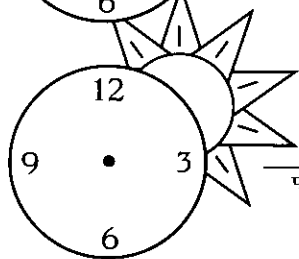

_____ часов дня, перед ленчем

_____ количество единиц


_____ количество единиц

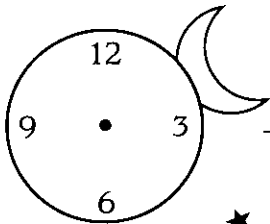

_____ часов вечера, перед обедом
или ужином

_____ количество единиц

_____ количество единиц

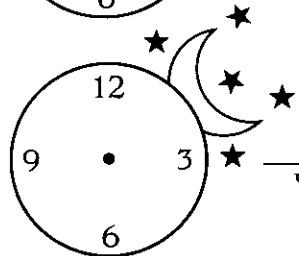

_____ часов вечера перед сном

_____ количество единиц

_____ количество единиц

**BECTON
DICKINSON**




BECTON DICKINSON CONSUMER PRODUCTS
Becton Dickinson and Company
Franklin Lakes, New Jersey 07417-1883

B-D, GETTING STARTED and ULTRA-FINE are trademarks of Becton Dickinson and Company

B-D


GETTING STARTED[®]

Drawing and Self-Injecting Insulin

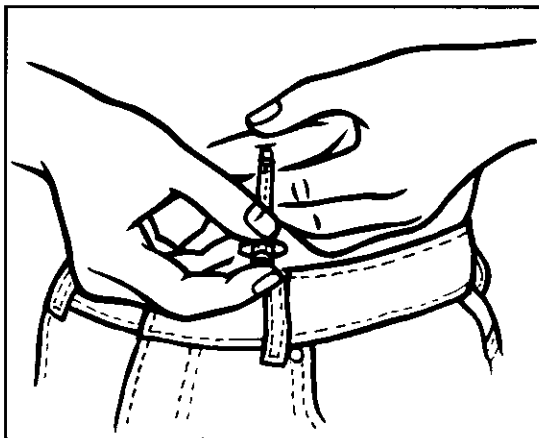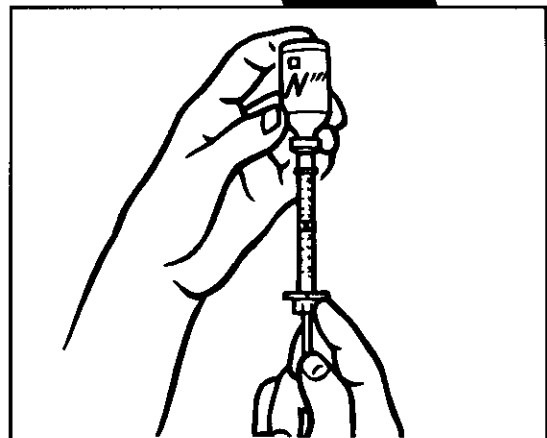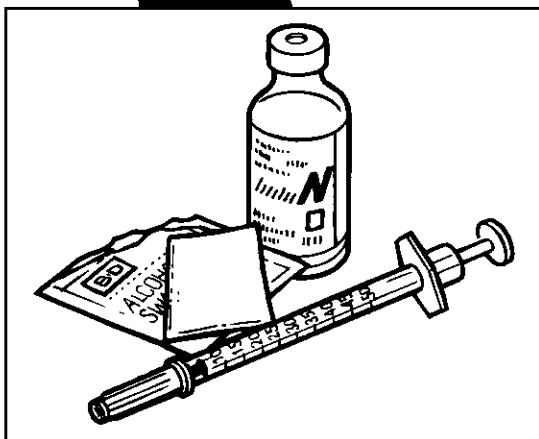


**BECTON
DICKINSON**


1. Get supplies

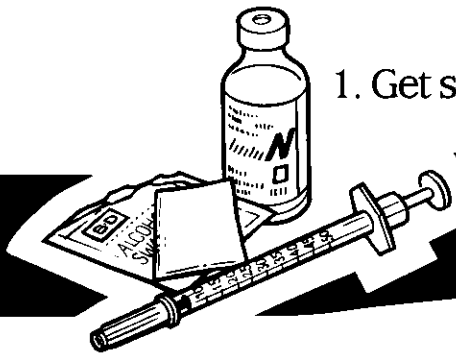


2. Wash hands

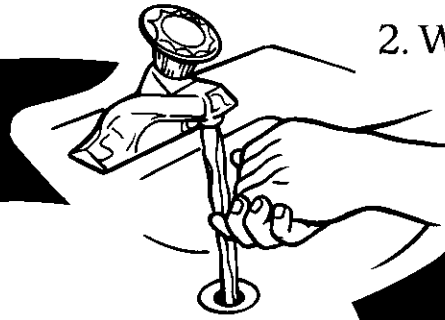


3. Roll bottle

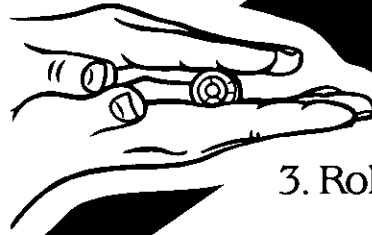


4. Wipe top of bottle with B-D Alcohol Swab

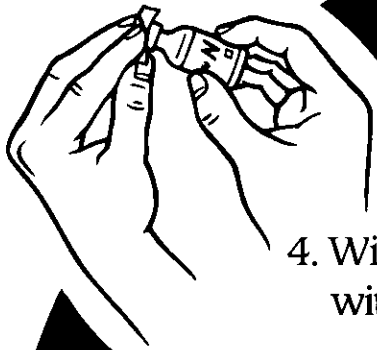



7. Push plunger up

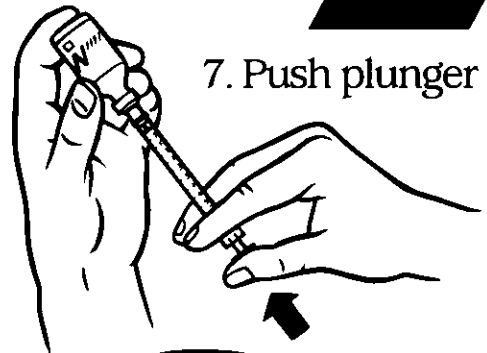


5. Pull plunger down to ___ units

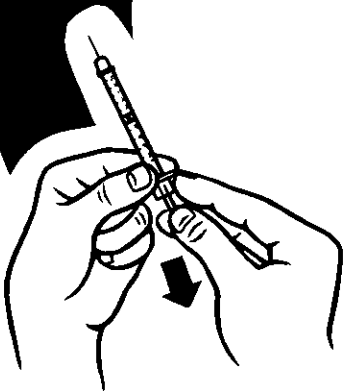


6. Push needle into bottle

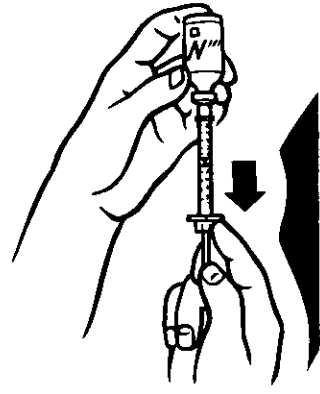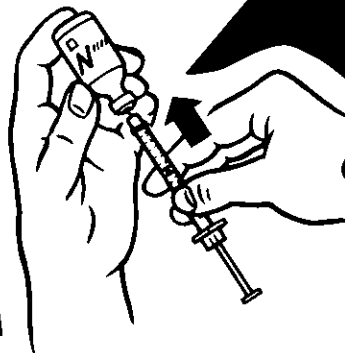

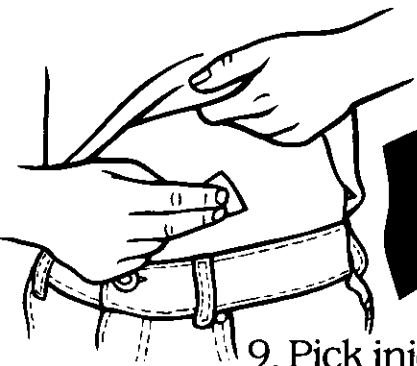


9. Pick injection site
Wipe with B-D Alcohol Swab

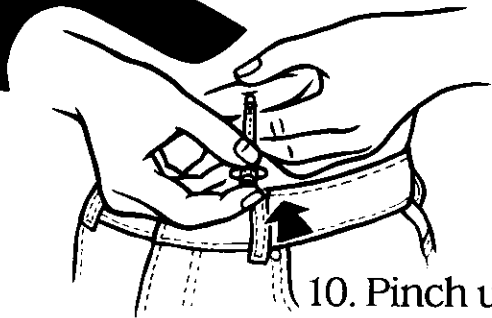


10. Pinch up skin
Push needle into skin
Push plunger in


8. Pull plunger down
to ____ units

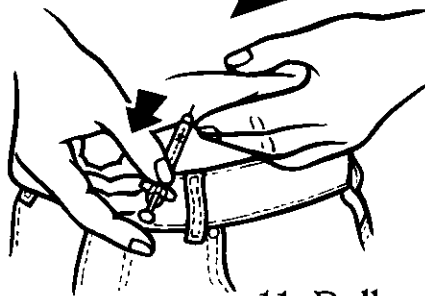


11. Pull needle out

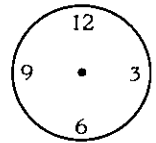

12. Dispose syringe safely
Check your town rules

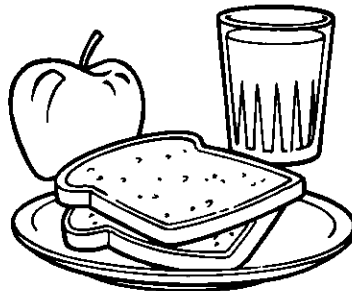

13. Eat ____ later

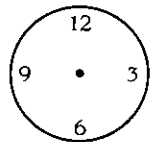


Insulin Dose for:




Name: _____ Today's date: _____

My educator is: _____ Telephone number: _____

Syringe size: **B-D** ULTRA-FINE® 3/10 cc 1/2 cc 1 cc

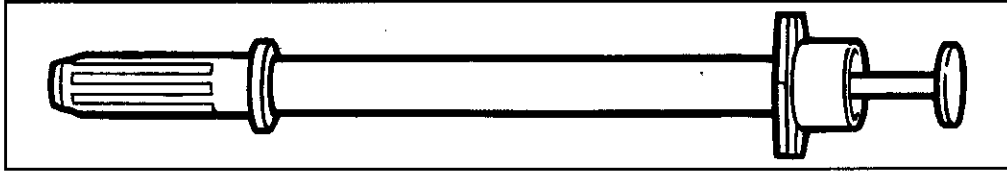


My insulin is: _____
(Brand Name)

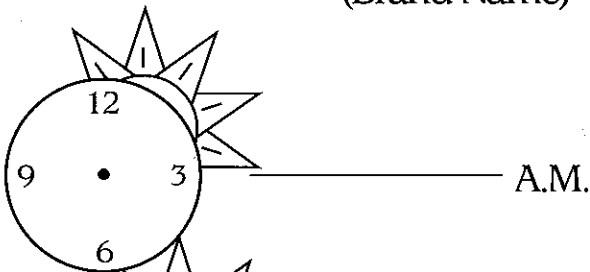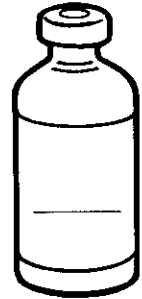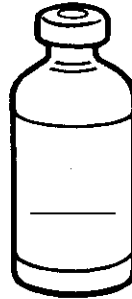

_____ A.M.

_____ units


_____ units

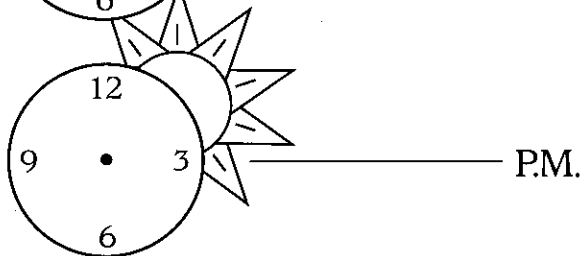

_____ P.M.

_____ units


_____ units

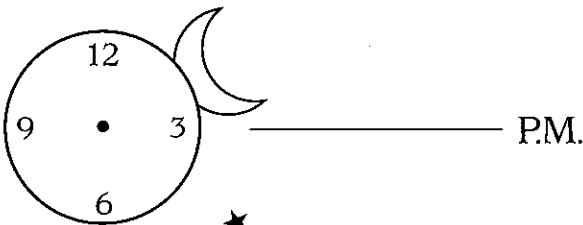

_____ P.M.

_____ units

_____ units

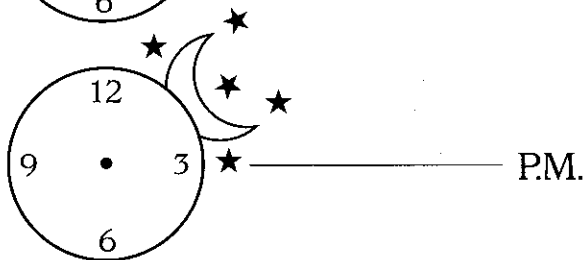

_____ P.M.

_____ units

_____ units

**BECTON
DICKINSON**

BECTON DICKINSON CONSUMER PRODUCTS
Becton Dickinson and Company
Franklin Lakes, New Jersey 07417-1883

B-D, ULTRA-FINE and GETTING STARTED are trademarks of Becton Dickinson and Company.

©1997 Becton Dickinson and Company

Printed in U.S.A.