ACCU-CHEK[®] *Nano*

BLOOD GLUCOSE METER / MEDIDOR DE GLUCEMIA

**Owner's Booklet for Self-Testing Only /
Manual del usuario únicamente para
autocontrol**

Dear ACCU-CHEK System Owner,

Thank you for choosing the ACCU-CHEK Nano SmartView system!

Congratulations on your decision to take control of your diabetes. We've designed the new ACCU-CHEK Nano SmartView system with comfort, convenience, and control in mind to help make living with diabetes a little easier.

This booklet will help you get the most from your ACCU-CHEK Nano SmartView system.

To start testing quickly, refer to the Quick Start Guide.

If you have questions, we are here to help. Just contact the ACCU-CHEK Customer Care Service Center toll-free at 1-800-858-8072. We offer assistance 24 hours a day, 365 days a year in many languages. You can also visit accu-chek.com for diabetes management tools and product demonstrations.

Thanks again for choosing the ACCU-CHEK Nano SmartView system.

WARNING

Before using the system, please read the Important Safety Information section on the following pages.

Contents

Introduction	7
The ACCU-CHEK Nano SmartView System	7
Important Safety Information	9
About Testing Yourself or Others	10
Before You Start Testing	11
Chapter 1: Understanding Your New System	13
Using the ACCU-CHEK Nano SmartView System	15
Chapter 2: Control Testing	17
Why Perform a Control Test	17
About the Control Solution	18
Performing a Control Test	19
Understanding Out-of-Range Control Test Results	23
Chapter 3: Testing Your Blood Glucose	25
Using the ACCU-CHEK FastClix Lancing Device	25
Inserting a Lancet Drum	27
Preparing the Lancing Device for Fingertip Testing	28
Performing a Blood Glucose Test with Blood from Your Fingertip	29
Changing the Lancet Drum	34
Performing a Blood Glucose Test with Blood from Your Palm (Alternate Site Testing)	36
Marking Test Results and Setting the Post-Meal Test Reminder	44
Unusual Test Results	46
Symptoms of Low or High Blood Glucose	47
Comparing Your Meter Result to a Laboratory Result	48

Chapter 4: Meter Memory, Setup, and Data Transfer	49
Memory	49
Viewing Test Results.....	50
Meter Setup.....	52
Setting the Time and Date	53
Setting the Beeper On/OFF	55
Setting the Post-Meal Test Reminder	57
Setting the Test Reminders.....	59
Setting the Hypoglycemic (Hypo) Alert	62
Transferring Results to a Computer	65
Chapter 5: Cleaning and Disinfecting the Meter and Lancing Device	67
Cleaning and Disinfecting the Meter	69
Cleaning and Disinfecting the Lancing Device	74
Chapter 6: Maintenance and Troubleshooting	79
Meter Maintenance.....	79
Changing the Batteries	80
Display and Error Messages	81
Chapter 7: Technical Information.....	87
Product Limitations.....	87
Specifications	87
Product Safety Information	89
Travel Documentation.....	89
Disposing of the Meter, Test Strips, Lancets, and Batteries	90
Explanation of Symbols.....	91
Warranty	92

Limited License	94
Additional Supplies	96
Index.....	99

Introduction

The ACCU-CHEK Nano SmartView System

The ACCU-CHEK Nano SmartView blood glucose monitoring system is intended to be used for the quantitative measurement of glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips or palm. The ACCU-CHEK Nano SmartView blood glucose monitoring system is intended to be used by a single person and should not be shared.

The ACCU-CHEK Nano SmartView blood glucose monitoring system is intended for self testing outside the body (in vitro diagnostic use) by people with diabetes at home as an aid to monitor the effectiveness of diabetes control. The ACCU-CHEK Nano SmartView blood glucose monitoring system should not be used for the diagnosis of or screening of diabetes or for neonatal use. Alternate site testing should be done only during steady-state times (when glucose is not changing rapidly).

The ACCU-CHEK SmartView test strips are for use with the ACCU-CHEK Nano blood glucose meter to quantitatively measure glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips or palm.

This system is intended to be used by a single person and should not be shared.

NOTE

The term “blood glucose” is used when referring to “blood sugar” in this booklet.

The ACCU-CHEK Nano SmartView blood glucose monitoring system includes:

- **ACCU-CHEK Nano meter with batteries**
- **ACCU-CHEK SmartView test strips***
- **ACCU-CHEK SmartView control solution***
- **ACCU-CHEK FastClix lancing device***
(with a black cap for fingertip testing and an AST cap** for alternate site testing)
- **ACCU-CHEK FastClix lancet drum***

*may be sold separately

**available upon request from the ACCU-CHEK Customer Care Service Center

Need Help?

Just contact the ACCU-CHEK Customer Care Service Center toll-free at 1-800-858-8072. We offer assistance 24 hours a day, 365 days a year in many languages. You can also visit accu-chek.com for diabetes management tools and product demonstrations.

Please complete the warranty card and mail it, so you receive the best customer service possible and product update news.

Important Safety Information

WARNING

- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.^{1,2}
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2}
- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.³
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected. For instructions on how to clean and disinfect the meter and lancing device, see Chapter 5, Cleaning and Disinfecting the Meter and Lancing Device.
- Wash hands thoroughly before and after handling the meter, lancing device, or test strips.

About Testing Yourself or Others

WARNING

- If you are very dehydrated or urinating frequently, you may get an inaccurate test result. If you think you are dehydrated, contact your healthcare professional right away.
- **DO NOT CHANGE YOUR TREATMENT BASED ON A SINGLE TEST RESULT THAT DOES NOT MATCH HOW YOU FEEL OR IF YOU BELIEVE THAT YOUR TEST RESULT COULD BE INCORRECT.**
- Some people with diabetes do not experience symptoms of low blood glucose (hypoglycemia). Others, such as children or people who are unconscious or have certain disabilities, may not be able to communicate their symptoms to caregivers. For these reasons, do not change any treatment without first talking to a healthcare professional.
- It is always a good idea to have a back-up testing method available. Failure to test could cause a delay in treatment decisions and lead to a serious medical condition. Examples of back-up testing methods include a back-up meter or testing by a laboratory. Ask your healthcare professional or pharmacist about other possible back-up methods.
- If your blood glucose result does not match how you feel and you have followed the instructions in this Owner's Booklet, follow your healthcare professional's instructions, or contact your healthcare professional.

NOTE

- Perform a control test when you open a new box of test strips or if you think that a test result is incorrect. Performing a control test lets you know that the meter and test strips are working properly.
- Refer to the test strip and control solution package inserts for additional health-related information.

Before You Start Testing

About the Meter and Test Strips

WARNING

- Carefully read and follow the instructions in the Owner's Booklet and package inserts for the test strips and control solution. It is very important to follow the instructions in order to prevent a wrong result or improper treatment.
- Inspect the container of test strips before using the test strips for the first time. If you see any damage to the container cap or if anything prevents the cap from closing properly, do not use the test strips. Contact the ACCU-CHEK Customer Care Service Center. Damaged test strips can cause inaccurate results, which could lead to improper treatment.
- The meter, test strips, and control solution are only for use outside the body (in vitro). Do not eat the test strips. Do not swallow or inject the control solution or use the control solution for any purpose other than testing the ACCU-CHEK Nano SmartView system.

NOTE

- Set the time and date on the meter before you begin testing.
- Although you always apply fresh capillary whole blood to the test strip, the system has been calibrated to deliver plasma-like values for easier comparison to lab results.

Special Information for Caregivers

- Do not use this device to measure blood glucose in people who are experiencing cardiovascular collapse (severe shock) or decreased peripheral blood flow.
- Consult your healthcare professional to determine if it is appropriate for your child to be taught how to use the meter system or any other medical products.

Chapter 1: Understanding Your New System

Right Arrow and Left Arrow Buttons – Press to enter memory, adjust settings, and scroll through test results.

Display – Shows results, messages, and test results stored in memory.

Test Strip Slot – Insert test strip here.

Front View

Infrared (IR) Window – Transfers data from the meter to a computer.

Power/Set Button – Turns the meter on or off and sets options.

Battery Drawer – Pull out the battery drawer to replace batteries, when needed.

Back View

Batteries – Insert into battery drawer with (+) side facing down.

Golden End – Insert this end of the test strip into the meter.

Test Strip

Yellow Window – Touch blood drop or control solution here.

Test Strip Container

Control Solution Bottle

AST Cap (for obtaining blood from sites other than your fingertip)

Lancet Drum

Side View

Power/Set Button

Infrared (IR) Window

Top View

Left Arrow Button

Comfort Dial with Depth Selection

Lancet Counter

Release Button

Cap

Depth Indicator

Lever (for advancing to new lancet)

Right Arrow Button

Lancing Device

NOTE

Some items may be sold separately.

Using the ACCU-CHEK Nano SmartView System

- Only use ACCU-CHEK SmartView test strips.
- Refer to the test strip package insert for test strip storage and system operating conditions.
- Store the unused test strips in their original container with the cap tightly closed.
- Close the container tightly immediately after removing a test strip. Moisture can damage the test strips and produce incorrect results.
- Do not remove test strips from the test strip container and put them into another container, such as a plastic bag, pocket, purse, wallet, etc.
- Discard the test strips if they are past the **Use By** date printed on the test strip container. If the **Use By** date is missing or cannot be read, do not use the test strips. Contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
- Use the test strip immediately after removing it from the container.
- Do not apply blood or control solution to the test strip before inserting it into the meter. If a result appears before applying blood or control solution, do not act on that result.
- Do not reuse test strips. Once control solution or blood has been applied to a test strip, discard it. If a retest is necessary, use a new test strip.
- Perform a control test every time you open a new box of test strips.

WARNING

- **DO NOT** expose test strips to heat, moisture, or humidity. Temperatures outside the required range, as well as moisture and humidity, can damage the test strips and lead to inaccurate results.
- **DO NOT** bend, cut, or alter the test strips.
- **DO NOT** get dirt, food, or other material on the test strip.

Not following these precautions can lead to inaccurate results.

Chapter 2: Control Testing

Why Perform a Control Test

Performing a control test lets you know that the meter and test strips are working properly. You should perform a control test when:

- You open a new box of test strips
- You left the test strip container open or you think the test strips have been damaged
- The test strips were stored in extreme temperatures and/or humidity
- You want to check the meter and test strips
- You dropped the meter
- Your test result does not match how you feel
- You want to check if you are testing correctly

About the Control Solution

- Only use the ACCU-CHEK SmartView control solution.
- Close the control solution bottle tightly after use.
- Write the date you open the control solution bottle on the bottle label. The control solution must be discarded 3 months from the date the bottle was opened (discard date) or on the **Use By** date on the bottle label, whichever comes first.
- Do not use control solution that is past the **Use By** or discard date.
- Refer to the control solution package insert for control solution storage conditions.
- The meter automatically recognizes the difference between the control solution and blood.
- The control results are not displayed in memory.
- The control solution can stain fabric. Remove stains by washing with soap and water.

Performing a Control Test


You need the meter, a test strip, and control solution.

1. To make sure the display is working properly, turn off the meter, then press and **hold**  to see the complete display. All segments should be clear and look exactly like the picture. If any segments are missing from the display, there may be a problem with the meter. Contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

NOTE

Control solution is available for purchase. To order the control solution, talk to your pharmacist or visit accu-chek.com to order online. The meter automatically recognizes the difference between the control solution and blood. The control results are not displayed in memory.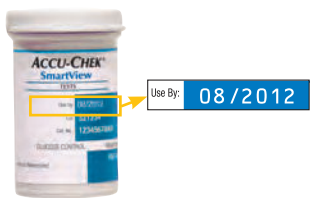


2. Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.

3. Insert the test strip into the meter in the direction of the arrows.

4. Place the meter on a flat surface, such as table.

5. Remove the control bottle cap and wipe the tip of the bottle with a tissue.

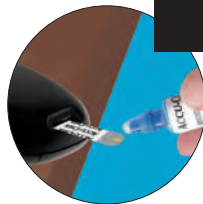6. Squeeze the bottle until a tiny drop forms at the tip. Touch the drop to the **front edge** of the yellow window of the test strip. **Do not put control solution on top of the test strip.** When you see  flash, you have enough control solution in the test strip.

(in range example)

7. Wipe the tip of the bottle with a tissue, then cap the bottle tightly. The result and the control bottle symbol appear on the display.

8. The control result and **OK** alternate on the display if the result is in range. The range is printed on the test strip container label. The control result and **Err** alternate on the display if the result is not in range. Remove and discard the used test strip.

Understanding Out-of-Range Control Test Results

WARNING

The control range applies only to a control result. It only indicates that the test strips and meter are working properly. Do not use a control result to interpret blood glucose results.

If the control result is outside the acceptable range (printed on the test strip container), do not use the meter until you solve the problem. Check this list to help solve the problem:

Troubleshooting Checks	Action
1. Are the test strips or control solution past the Use By or discard date?	If either is past the Use By or discard date, throw it away.
2. Did you wipe the tip of the control solution bottle with a tissue prior to use?	Wipe the tip of the bottle with a tissue. Repeat the control test with a new test strip and a fresh drop of control solution.
3. Were the test strip container and control solution bottle always closed tightly?	If you think either may have been uncapped for some time, replace the test strips or control solution.
4. Was the test strip used immediately after removing it from the container?	Repeat the control test with a new test strip and a fresh drop of control solution.
5. Were the test strips and control solution stored in a cool, dry place?	Repeat the control test with a properly stored test strip or control solution.

Troubleshooting Checks	Action
6. Did you follow all of the testing instructions?	Read Chapter 2, Control Testing, and test again. If you still have problems, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
7. If you are still unsure of the problem ...	Repeat the control test with a new test strip. If you still have problems, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Chapter 3: Testing Your Blood Glucose

Using the ACCU-CHEK FastClix Lancing Device

WARNING

- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.^{1,2}
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2}
- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.³
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected. For instructions on how to clean and disinfect the meter and lancing device, see Chapter 5, Cleaning and Disinfecting the Meter and Lancing Device.
- Wash hands thoroughly before and after handling the meter, lancing device, or test strips.
- You must not insert the lancet drum into the lancing device and simultaneously press the release button or hold the lancing device with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

NOTE

- Testing with blood from alternate sites requires the AST cap. This is explained later in the chapter.
- The best depth setting is the lowest number that lets you get enough blood for a test. Try different settings to find the one that is right for you.
- ALWAYS use a new, sterile lancet each time you test. Never reuse a lancet that has already been used.
- NEVER share your lancing device with anyone.

26

The black cap is for fingertip testing only.

The AST cap is for testing other approved sites on the body, not the fingertip.*

A new drum is dark gray with a white end.

A used drum has a visible red stripe on the white end. You cannot insert a used drum into the lancing device.

*available upon request from the ACCU-CHEK Customer Care Service Center at 1-800-858-8072

Inserting a Lancet Drum

You must first load the drum into the lancing device to get it ready for use.

WARNING

You must not insert the lancet drum into the lancing device and simultaneously press the release button or hold the lancing device with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

1. Remove the cap by pulling it straight off. Do not twist. The easiest way to do this is to place your thumb to the side of the notch as seen in the illustration.

2. Insert a new drum, white end first, until it clicks firmly into place.

3. Slide the cap on until it stops by aligning the notch on the cap with the notch on the device.

4. You are now ready to use the first lancet. The lancet counter on the device shows a number 6, meaning you have six new lancets remaining.

NOTE

Once the drum is inserted into the lancing device, do not remove the drum until completely used. The drum cannot be reinserted.

Preparing the Lancing Device for Fingertip Testing

WARNING

Use a new lancet for each fingerstick to avoid infection.

28

1. Make sure the black cap is on the lancing device. Do not use the AST cap to perform a fingertip test.
2. Adjust the lancet depth by turning the Comfort Dial. The depth indicator shows the current depth selection. The higher the number, the deeper the penetration. If you have soft skin, we suggest starting at a depth of 2. If your skin is calloused or thick, try a higher setting.
3. Set the lancing device aside until you are ready to perform a fingerstick.

Performing a Blood Glucose Test with Blood from Your Fingertip

Before you perform your first blood glucose test, set up the meter correctly and perform a control test. You need the meter, a test strip, and a lancing device with a drum loaded to perform a blood test.

WARNING

- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.^{1,2}
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2}
- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.³
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected. For instructions on how to clean and disinfect the meter and lancing device, see Chapter 5, Cleaning and Disinfecting the Meter and Lancing Device.
- Wash hands thoroughly before and after handling the meter, lancing device, or test strips.

Use By: 08/2012

1. To make sure the display is working properly, turn off the meter, then press and **hold**  to see the complete display. All segments should be clear and look exactly like the picture. If any segments are missing from the display, there may be a problem with the meter. Contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
2. Wash your hands with warm soapy water and dry thoroughly. Dirty or wet hands could affect results.
3. Prepare the lancing device.
4. Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.

5. Insert the test strip into the meter in the direction of the arrows. The meter beeps.

6. The meter beeps again and a flashing blood drop symbol appears.


7. Place the lancing device firmly against the side of your fingertip. Remember that the opening where the lancet comes out is not in the center of the cap. Press the release button on the end of the device all the way down to prick the finger. The ACCU-CHEK FastClix lancing device is primed and pricks the finger in a single push of the release button.

WARNING

If mg/dL does not appear with the test result, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072. Use of the wrong unit of measure may cause misinterpretation of your actual blood glucose level, and may lead to incorrect treatment.

32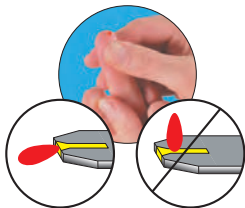

8. Gently squeeze your finger to assist the flow of blood. This helps you get a blood drop.


9. Touch the blood drop to the **front edge** of the yellow window of the test strip. **Do not put blood on top of the test strip.** The meter beeps and  flashes when there is enough blood in the test strip. The result appears on the display.

10. If you want to mark the test result with a pre-meal, post-meal, or general marker, leave the test strip in the meter. See Chapter 3, Marking Test Results and Setting the Post-Meal Test Reminder. Otherwise, discard the used test strip. After a successful test, the meter turns itself off 5 seconds after the test strip is removed.

11. **ALWAYS** use a new, sterile lancet each time you test. **NEVER** reuse a lancet that has already been used. To advance to the next lancet, slide the lever forward and then slide it back all the way.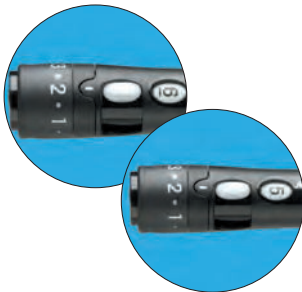

12. The lancet counter decreases one number. The window shows the number of lancets remaining. For safety reasons, once you advance to a new lancet, you cannot go back to a used lancet.

13. Wash hands thoroughly with soap and water.

Changing the Lancet Drum

When you have used the sixth and last lancet, you should change the drum.

34

1. Take the black cap off the lancing device by pulling it straight off. It is not a twist cap. The easiest way to do this is to place your thumb to the side of the notch as seen in the illustration.

2. Hold the drum between your thumb and index finger and pull it straight out of the lancing device. A red stripe is visible on the white part, indicating that the drum has been used.

NOTE

A used drum cannot be reinserted into the lancing device.

3. Throw away the old drum. Always dispose according to local regulations.

4. Insert a new drum, white end first, until it clicks firmly into place.

NOTE


Once the drum is inserted into the lancing device, do not remove the drum until completely used. The drum cannot be reused.

Performing a Blood Glucose Test with Blood from Your Palm (Alternate Site Testing)

WARNING

- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.^{1,2}
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2}
- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.³
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected. For instructions on how to clean and disinfect the meter and lancing device, see Chapter 5, Cleaning and Disinfecting the Meter and Lancing Device.
- Wash hands thoroughly before and after handling the meter, lancing device, or test strips.

You have the option of obtaining a blood sample from other sites on your body besides the fingertip. Alternate sites include two palm testing sites. The two palm testing sites are the fleshy areas under the thumb (thenar) and under the little finger (hypothenar). Blood obtained from the fingertip can be used at any time to test blood glucose. If blood from an alternate site is used, there are certain times when testing is not appropriate (see next section). This is because your blood glucose level changes faster in your fingertip than in the alternate sites. These differences may cause you to make the wrong therapeutic decision, producing adverse health effects. Please read the following section before you try testing from alternate sites.

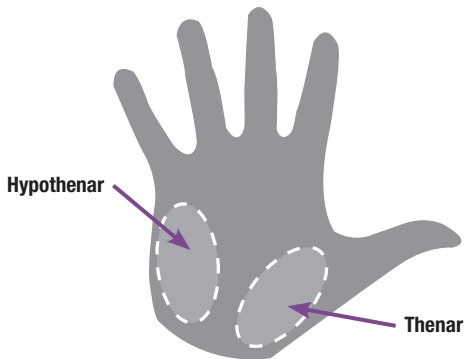**NOTE**

Talk to your healthcare professional before you begin using alternate test sites.

WARNING

Do not change your treatment because of just one test result.

Do not use an AST measurement to calibrate a continuous glucose monitoring (CGM) device or to make insulin dosing calculations.

NEVER ignore symptoms of low or high blood glucose.

If your blood glucose result does not match how you feel, perform a fingertip test to confirm your result. If the fingertip result still does not match how you feel, contact your healthcare professional.


Tests from the palm may be performed:

- Immediately before a meal
- Fasting

Tests from the palm may NOT be performed:

- 2 hours or less after eating
- After exercising
- If you are sick
- If you think your blood glucose is low
- If you often do not notice when your blood glucose is low
- During peak action time of short-acting insulin or rapid-acting insulin analogues
- Up to 2 hours after injecting a short-acting insulin or a rapid-acting insulin analogue

You need the meter, a test strip, a lancing device with a drum loaded, and the AST cap (available upon request from the ACCU-CHEK Customer Care Service Center).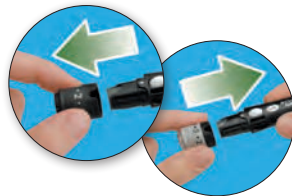

1. To make sure the display is working properly, turn off the meter, then press and **hold**  to see the complete display. All segments should be clear and look exactly like the picture. If any segments are missing from the display, there may be a problem with the meter. Contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
2. Wash your hands and the test site with warm, soapy water and dry thoroughly.
3. Remove the black cap from the lancing device by pulling it straight off. It is not a twist cap. The easiest way to do this is to place your thumb to the side of the notch as seen in the illustration.
4. Put the AST cap on by lining up the notched sections on the cap and the lancing device.

5. Dial to the depth selection appropriate for the testing site. Set the lancing device aside until later in the test.

6. Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.

7. Insert a test strip into the meter in the direction of the arrows.

NOTE

We suggest starting at a depth setting of 3 for palm testing. Once you perform a successful test, you may want to find the lowest depth setting that provides enough blood with the least amount of pain.

8. When the blood drop symbol flashes, obtain a blood drop from the site.

9. Press the opening of the AST cap firmly against a fleshy area on the alternate site. Press the lancing device up and down in a slow pumping motion to assist the flow of blood. Remember that the opening where the lancet comes out is not in the center of the cap.

10. Keep steady pressure on the area and press the release button on the end of the device all the way down to prick the site. Do not be afraid to apply pressure to the area with the lancing device to assist the flow of blood. Continue to press the cap against the area for a few seconds to allow blood to come to the surface.

NOTE


To get a good blood drop:

- Position hand on a table or surface below heart level.
- Rub the skin prior to lancing to increase blood flow.
- If the blood drop is too small, reapply pressure with the AST cap. DO NOT squeeze the AST site.

WARNING

If mg/dL does not appear with the test result, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072. Use of the wrong unit of measure may cause misinterpretation of your actual blood glucose level, and may lead to incorrect treatment.

42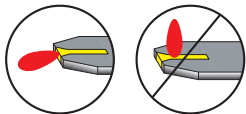


11. Touch the blood drop to the **front edge** of the yellow window of the test strip. **Do not put blood on top of the test strip.**


12. When you see  flash, there is enough blood in the test strip.

13. The result appears on the display. If you want to mark the test result with a pre-meal, post-meal, or general marker, leave the test strip in the meter. See Chapter 3, Marking Test Results and Setting the Post-Meal Test Reminder. Otherwise, discard the used test strip.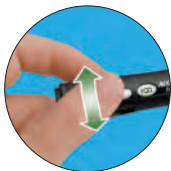

14. ALWAYS use a new, sterile lancet each time you test. NEVER reuse a lancet. To load the next lancet, slide the lever forward (until it stops), then slide it back all the way.

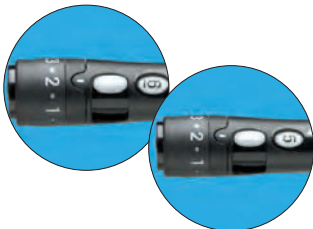15. The lancet counter decreases one number. For safety reasons, once you advance to a new lancet, you cannot go back to a used lancet.

16. Wash hands thoroughly with soap and water.

NOTE

Having trouble testing from alternate sites?

If you experience difficulty obtaining a sufficient blood drop while testing from an alternate site, please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072 for assistance. We will guide you through the testing procedure step by step. If you continue to have difficulty, we can also offer you, free of charge, our ACCU-CHEK Softclix lancing device. Because the ACCU-CHEK Softclix lancet has a thicker diameter needle than the ACCU-CHEK FastClx lancet, you may have better success when trying to test from alternate sites.

Marking Test Results and Setting the Post-Meal Test Reminder

If you wish, you can mark your test result with:

Pre-Meal Marker

Pre-Meal Marker with Post-Meal Test Reminder

Post-Meal Marker

General Marker

44

- Marking results with a **pre-meal** or **post-meal marker** provides more information about your blood glucose results to help you and your healthcare professional in the management of your diabetes.
- When a pre-meal result is marked with a **post-meal test reminder**, the meter beeps 1 or 2 hours after you test to remind you to do a post-meal test.
- You might want to use the **general marker** to mark an event such as an alternate site test result or exercise.

When you review the results in memory, these markers can help you remember what was different about the result.

Here is how to mark a test result and initiate a post-meal test reminder:

1. Perform a test. **Do not remove the test strip.**
2. Press and release  to toggle through the test result markers and post-meal test reminder.

Pre-Meal Marker

Pre-Meal Marker with Post-Meal Test Reminder

Post-Meal Marker

General Marker

3. When the test result marker (, , or ) or pre-meal marker with post-meal test reminder () that you wish to select appears on the display, remove the test strip from the meter.

- If you select the post-meal test reminder, the meter beeps 1 or 2 hours after the pre-meal test to remind you to do a post-meal test.
- If you select a test result marker, it is automatically stored in memory.

NOTE

The post-meal marker automatically appears with the test result if a test is performed 15 minutes before or after the post-meal test reminder is programmed to beep. Set the post-meal test reminder time to 1 or 2 hours in the set-up mode.

Unusual Test Results

If your blood glucose result does not match how you feel, follow these steps:

1. Perform a control test. See Chapter 2, Control Testing.
2. Repeat the blood glucose test. See Chapter 3, Performing a Blood Glucose Test with Blood from Your Fingertip.
3. If your blood glucose result still does not match how you feel, follow your healthcare professional's instructions or call your healthcare professional immediately.

NOTE

Always follow your healthcare professional's instructions. For example, if your healthcare professional has advised you to immediately treat a low blood glucose result (such as by eating something), then do that first.

Symptoms of Low or High Blood Glucose

WARNING

The meter is designed to provide a numerical value for blood glucose in the range of 20–600 mg/dL. If you receive a numerical value below 20 mg/dL or over 600 mg/dL and it does not match how you feel, contact the ACCU-CHEK Customer Care Service Center toll-free at 1-800-858-8072.

Being aware of the symptoms of low or high blood glucose can help you understand your test results and decide what to do if they seem unusual.

Low blood glucose (hypoglycemia): Symptoms of hypoglycemia may include, but are not limited to, anxiety, shakiness, sweating, headache, increased hunger, dizziness, pale skin color, sudden change in mood or irritability, fatigue, difficulty concentrating, clumsiness, palpitations, and/or confusion.

High blood glucose (hyperglycemia): Symptoms of hyperglycemia may include, but are not limited to, increased thirst, frequent urination, blurred vision, drowsiness, and/or unexplained weight loss.

If you are experiencing any of these symptoms, or other unusual symptoms, test your blood glucose from the fingertip. If your blood glucose result is displayed as LO or HI, follow your healthcare professional's instructions or contact your healthcare professional immediately. If your blood glucose result does not match how you feel, follow the steps under Chapter 3, Unusual Test Results.

Comparing Your Meter Result to a Laboratory Result

A common question is how the blood glucose results on the meter compare to the laboratory results. Your blood glucose can change quickly, especially after eating, taking medication, or physical activity. If you test yourself in the morning, then go to your healthcare professional's office for a blood glucose test, your results will probably not match, even if you are fasting. This is typically not a problem with the meter, it just means that time has elapsed and your blood glucose has changed.

If you want to compare your meter result to the laboratory result, **you must be fasting**. Take the meter to your healthcare professional's office and test yourself by fingerstick within 5 minutes of having blood drawn from your arm by a healthcare professional. Keep in mind that the laboratory uses different technology than the meter and that blood glucose meters for self-testing generally read somewhat lower than the laboratory result.

If you are fasting and you do a fingerstick test within 5 minutes of having your blood drawn, here are the general guidelines to compare the meter result to the laboratory result:

- If your blood glucose is below 75 mg/dL, your results generally should fall within ± 15 mg/dL of the laboratory result.
- If your blood glucose is equal to or over 75 mg/dL, your results generally should fall within ± 20 % of the laboratory result.

Chapter 4: Meter Memory, Setup, and Data Transfer

Memory

Storing Blood Glucose and Control Test Results

The meter automatically stores up to 500 blood glucose test results with the time and date of the test and any test markers. Results can be reviewed at any time. Test results are stored from the newest to the oldest, so set the time and date correctly in the meter. Having the correct time and date setting helps ensure appropriate interpretation of stored blood glucose results by you and your healthcare team.

NOTE

- The memory is not lost when the batteries are replaced; however, confirm that the time and date are still correct. See Chapter 4, Setting the Time and Date.
- Once 500 blood glucose results or 20 control results are in memory, adding a new result causes the oldest one to be deleted.
- If more than 500 tests are performed within a 90-day period, only the most recent 500 results are included in the 90-day average.
- Press and **hold** ◀ or ▶ to scroll through the results faster.
- The control results are stored in memory, but cannot be reviewed on the meter. They are not included in the 7-, 14-, 30-, and 90-day averages. To view stored control results, first transfer them to a compatible software application. For product availability, please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Viewing Test Results


WARNING

DO NOT CHANGE YOUR THERAPY BASED ON AN INDIVIDUAL TEST RESULT IN MEMORY. TALK TO YOUR HEALTHCARE PROFESSIONAL BEFORE CHANGING THERAPY BASED ON RESULTS IN MEMORY.

With the meter on or off, press and release ◀ or ▶ to enter memory. The most recent result appears on the display.

- To view previous results in order – press ◀.
- To look at the 7-, 14-, 30-, or 90-day averages – press ▶.
- To view pre-meal and post-meal 7-, 14-, 30-, and 90-day averages – continue to press ▶.

50

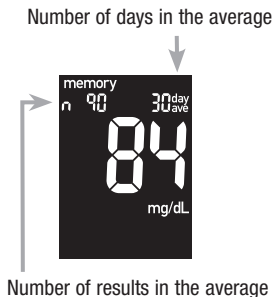Past Test Results

Press ◀ to view past results from newest to oldest.

General Average

Press ▶ to view 7-, 14-, 30-, and 90-day averages.

🍏 Pre-Meal Average

Continue to press ▶ to view 7-, 14-, 30-, and 90-day pre-meal averages.

🍷 Post-Meal Average

Continue to press ▶ to view 7-, 14-, 30-, and 90-day post-meal averages.

NOTE

Only test results that have been assigned a pre-meal (🍏) or post-meal (🍷) marker are included in pre-meal and post-meal averages. All blood glucose results are included in the general 7-, 14-, 30-, and 90-day averages.

Meter Setup

Using the Set-Up Mode

The following features can be customized as required:

Time and date – set the time and date.

Beeper – select **On** or **OFF**.

Post-meal test reminder – select 1 hour or 2 hours.

Test reminders – select **On** or **OFF**.

Hypo alert – select **On** or **OFF**. If you select **On**, select the blood glucose level for the alert.

Using the set-up mode is easy.

Here is a general overview of the function that each button performs in the set-up mode.

- **To enter the set-up mode:** With the meter on, press and **hold**  until **set-up** appears.
- **To set the feature you have chosen:** Press and release .
- **To exit the set-up mode:** Press and **hold**  until you see the flashing test strip symbol.
- **To adjust settings:** Press and release  or . To scroll faster, press and **hold**  or .

You are here

TIME / DATE
(hrs, min, am/pm,
month, day, year)

BEEPER
(on/off)

**POST-MEAL
TEST REMINDER**
(1 hr or 2 hr)

**TEST
REMINDERS**
(A-1, A-2, A-3, A-4)

**HYPO
ALERT**
(off, on, level)

Setting the Time and Date

1. Press and release  to turn on the meter. The flashing test strip symbol appears on the display.

2. Press and **hold**  until **set-up** appears on the display. The hour flashes.

3. Press and release  or  to decrease or increase the hour.

54

4. Press and release  to set the hour. The minutes flash.
5. Press and release  or  to adjust the minutes. Press and release  to set the minutes.

6. Repeat step 5 to set am/pm, month, day, and year.

7. To set up more options, press and release . To exit, press and **hold**  until the flashing test strip symbol appears on the display.

Setting the Beeper (🔊) On/OFF

The meter has the beeper preset to **On**. Setting the beeper to **OFF** does not affect test results.

The beeper is helpful because it prompts you:

- To apply blood or control solution to the test strip
- When enough blood or control solution is drawn into the test strip
- When the test is complete
- When a button is pressed
- When it is time to perform a test (if you set the test reminders or post-meal test reminder)
- If an error occurred while testing (even if the beeper is off, it still beeps for an error)

56

1. Press and release  to turn on the meter. The flashing test strip symbol appears on the display.
2. Press and **hold**  until **set-up** appears on the display.
3. Press and release  repeatedly until the flashing beeper symbol and **On** or **OFF** appear.
4. Press and release  or  to switch between **On** and **OFF**.
5. To set up more options, press and release . To exit, press and **hold**  until the flashing test strip symbol appears on the display.

Setting the Post-Meal Test Reminder (🍏🔔)



The post-meal test reminder:

- Beeps 1 or 2 hours after the blood glucose test to remind you to do a post-meal test
- Beeps every 2 minutes up to 3 times
- Turns off by inserting a test strip or pressing any button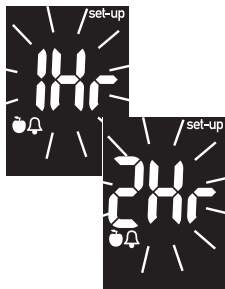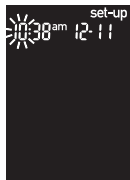

The post-meal test reminder time is preset on the meter to 2 hours. However, you can choose 1 or 2 hours in the set-up mode.

NOTE

- The beeper must be set to **On** for the test reminder to occur.
- If a test was performed within 15 minutes of a test reminder, the test reminder does not occur.
- If the meter is on at the test reminder time, the test reminder does not occur.
- Exposure to cold conditions may disable test reminders until the meter is turned on.

58

1. Press and release  to turn on the meter. The flashing test strip symbol appears on the display.
2. Press and **hold**  until **set-up** appears on the display.
3. Press and release  repeatedly until **set-up**, and the flashing **1Hr** or **2Hr** appear.
4. Press and release  or  to select **1Hr** or **2Hr**. Press and release  to set the option.

This display appears when a post-meal test reminder occurs.

Setting the Test Reminders (🔔)

Test reminders:

- Beep at the same time each day to remind you to perform a test.
- Beep every 2 minutes up to 3 times.
- Turn off by inserting a test strip or pressing any button.
- Are preset to **OFF**. You must turn them on to use this feature.

You can set up to 4 test reminders per day. If A-1, A-2, A-3, and A-4 are turned on, the meter is preset with the following times which can be adjusted as necessary.

A-1 8:00 am


A-2 12:00 pm (noon)

A-3 6:00 pm

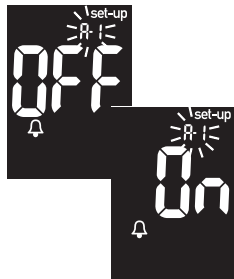
A-4 10:00 pm

NOTE

- If a test was performed within 15 minutes of a test reminder, the test reminder does not occur.
- If the meter is on at the test reminder time, the test reminder does not occur.
- Exposure to cold conditions may disable test reminders until the meter is turned on.

1. Press and release  to turn on the meter. The flashing test strip symbol appears on the display.
2. Press and **hold**  until **set-up** appears on the display.
3. Press and release  repeatedly until the bell symbol, **OFF**, **set-up**, and the flashing **A-1** appear.
4. Press and release  or  to switch between **On** and **OFF**. Press and release  to set the option.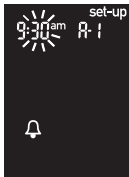

5. If **On** is selected, the hour flashes. **A-1** and the bell symbol remain on the display.
6. Press and release ◀ or ▶ to select the hour. Press and release ⏻ to set the hour.
7. The minutes flash. Press and release ◀ or ▶ to select 00, 15, 30, or 45. These are the only choices.
8. Press and release ⏻ to set the minutes. **am** or **pm** flashes on the display. Press and release ◀ or ▶ to switch between **am** and **pm**. Press and release ⏻ to set **am** or **pm**.
9. **Set-up, OFF**, and the bell symbol appear on the display with the next test reminder (**A-2**) flashing. You can either set a second test reminder or press and **hold** ⏻ until the flashing test strip symbol appears to exit.

NOTE

When you are setting the time for the test reminders, the bell symbol and **set-up** remain on the display.


Setting the Hypoglycemic (Hypo) Alert (⚠)

The hypo alert can be set to a level between 50–90 mg/dL to let you know when your blood glucose is possibly too low.

WARNING

- This function is no substitute for hypoglycemia training by your healthcare professional or diabetes team.
- Before you set the hypo alert, talk to your healthcare professional to help you decide what blood glucose level is your hypoglycemic level.

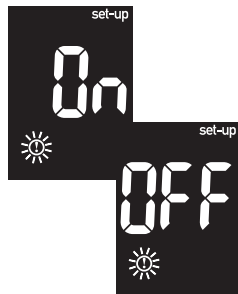1. Press and release to turn on the meter. The flashing test strip symbol appears on the display.
2. Press and **hold** until **set-up** appears on the display.
3. Press and release repeatedly until **set-up**, **OFF**, and the flashing appear.
4. Press and release or to switch between **On** and **OFF**. Press and release to set the option.

NOTE

The hypo alert is preset to OFF.

64

5. If **On** is selected, **set-up** appears and  flashes. The display shows 70 mg/dL.

6. Press and release  or  to select the level. Press and release  to set the level.

7. To exit, press and **hold**  until the flashing test strip symbol appears.


Transferring Results to a Computer

We offer a variety of software to help you transfer your results. For information on ACCU-CHEK software, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

You can transfer your stored results to a computer to track, identify patterns, and print.

Transferring Data Directly to a Computer Using Specialized Software and an Infrared Cable

1. Install the software according to the instructions.
2. To transfer the results to a computer, connect the infrared cable according to the instructions.
3. Run the software program and follow the instructions to transfer information. Make sure the software is ready to accept data from the meter.
4. With the meter off, press and **hold** both ◀ and ▶ until 2 arrows on the display alternately flash.

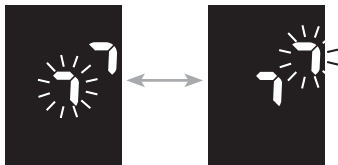5. Locate the infrared (IR) window on the top of the meter.
6. Locate the IR window on the infrared cable.
7. Place the meter on a flat surface. Point the two IR windows toward each other. They should be 1–4 inches apart.
8. Do not move the infrared cable or meter during the data transfer.
9. Follow the prompts on the software.
10. The software program may shut off the meter automatically when the data transfer is complete. Should this occur, follow the prompts on the display.

Infrared (IR) Window

NOTE

- If the data did not transfer successfully, try again. If you still have problems, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
- To make the most of the transfer feature, you must set the time and date in the meter correctly.

Chapter 5: Cleaning and Disinfecting the Meter and Lancing Device

WARNING

- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.^{1,2}
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2}
- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.³
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected.
- Wash hands thoroughly before and after handling the meter, lancing device, or test strips.

What is the difference between cleaning and disinfecting?

Cleaning is the removal of dirt from the meter or lancing device.³

Disinfecting is the removal of most, but not all, disease-causing and other types of microorganisms (bloodborne pathogens) from the meter or lancing device.³

When should the meter and lancing device be cleaned and disinfected?

Clean and disinfect the meter and lancing device:

- Once per week
- When blood gets on the meter or lancing device
- Before allowing anyone else to handle the meter or lancing device

NOTE

For technical assistance or questions on cleaning and disinfecting please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Cleaning and Disinfecting the Meter

To clean and disinfect without damaging the meter, follow these procedures carefully.

When to Clean and Disinfect the Meter

- Clean the meter to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the meter at least once per week and when blood is present on the surface of the meter.
- Clean and disinfect the meter before allowing anyone else to handle the meter. **Do not allow anyone else to use the meter on themselves for testing purposes.**

Approved Cleaning and Disinfecting Product

The following product has been approved for cleaning and disinfecting the meter:

- Super Sani-Cloth (EPA* reg. no. 9480-4)

*Environmental Protection Agency

Super Sani-Cloth can be purchased from the following sources:

- Amazon.com
- Officedepot.com
- Walmart.com

NOTE

If you notice any of the following signs of deterioration after cleaning and disinfecting your meter, please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072: Residue around buttons, clouding of display, or button malfunction.

What to Clean and Disinfect

The following parts of the meter should be cleaned and disinfected:

- The area **around** slots and openings (do not get any moisture in slots or openings)
- The meter display
- The entire meter surface

How to Clean and Disinfect the Meter

WARNING

- **DO NOT** clean or disinfect the meter while performing a blood glucose or control test.
- **DO NOT** get any moisture in slots or openings.
- **DO NOT** spray anything onto the meter.
- **DO NOT** immerse the meter in liquid.

Failure to follow these instructions will damage the meter and stop it from working properly.

WARNING

The effect of using more than one product interchangeably to clean and disinfect the meter has not been tested. Always use Super Sani-Cloth (EPA reg. no. 9480-4) to clean and disinfect the meter. Do not use any other cleaning or disinfecting solutions. Using solutions other than the Super Sani-Cloth could result in damage to the meter.

1. Wash hands thoroughly with soap and water.

2. Turn off the meter and wipe the entire meter surface with an approved cleaning and disinfecting product. Always use the same product for both cleaning and disinfecting.

3. Carefully wipe **around** the test strip slot and other openings. Make sure that no liquid enters any slot or opening.

4. Dry the meter with a soft cloth or gauze. Make sure that no solution is seen in any slot or opening.

5. For disinfecting the meter, get a new wipe or cloth. Repeat steps 2 and 3 following the manufacturer's instructions for disinfecting.

6. Wash hands thoroughly with soap and water.

NOTE

Roche has tested the approved product for a total of 520 cycles, which is equal to cleaning and disinfecting once per week for a 5 year period.

Cleaning and Disinfecting the Lancing Device

To clean and disinfect without damaging the lancing device, follow these procedures carefully.

When to Clean and Disinfect the Lancing Device

- Clean the lancing device to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the lancing device at least once per week to remove visible dirt or other material for safe handling.
- Clean and disinfect the lancing device before allowing anyone else to handle the lancing device, for instance, if you have someone assisting you. **Do not allow anyone else to use the lancing device.**

NOTE

- Do not throw away the lancing device cap after each use. Use the approved cleaning and disinfecting product on it.
- Remove the lancet drum before cleaning and disinfecting the lancing device.

Approved Cleaning and Disinfecting Product

The following product has been approved for cleaning and disinfecting the lancing device:

- Super Sani-Cloth (EPA* reg. no. 9480-4)

*Environmental Protection Agency

Super Sani-Cloth can be purchased from the following sources:

- Amazon.com
- Officedepot.com
- Walmart.com

NOTE

- If you notice any of the following signs of deterioration after cleaning and disinfecting your lancing device, please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072: Residue around buttons.
- You might observe a slight discoloration of the lancing device after multiple cleaning and disinfecting cycles. This does not affect the functionality of the device.

What to Clean and Disinfect

The following parts of the lancing device should be cleaned and disinfected:

- The entire lancing device surface
- The cap

How to Clean and Disinfect the Lancing Device

WARNING

DO NOT get any moisture into any openings.

Failure to follow these instructions may damage the lancing device and stop it from working properly.

1. Wash hands thoroughly with soap and water.


2. Wipe the entire surface of the lancing device and the inside of the lancing device cap with an approved cleaning and disinfecting product. Always use the same product for both cleaning and disinfecting.

3. Dry the lancing device and cap with a soft cloth or gauze. Make sure no solution is seen in any opening.

WARNING

The effect of using more than one product interchangeably to clean and disinfect the lancing device has not been tested. Always use Super Sani-Cloth (EPA reg. no. 9480-4) to clean and disinfect the lancing device. Do not use any other cleaning or disinfecting solutions. Using solutions other than the Super Sani-Cloth could result in damage to the lancing device.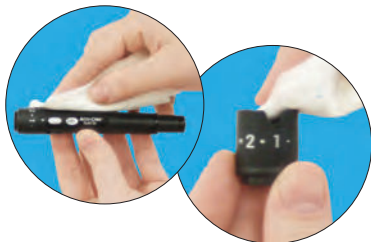

4. For disinfecting the lancing device, get a new wipe or cloth. Repeat step 2 following the manufacturer's instructions for disinfecting.

5. Wash hands thoroughly with soap and water.

NOTE

Roche has tested the approved product for a total of 520 cycles, which is equal to cleaning and disinfecting once per week for a 5 year period.

Chapter 6: Maintenance and Troubleshooting

Meter Maintenance

The meter needs little or no maintenance with normal use. It automatically tests its own systems every time you turn it on and lets you know if something is wrong. See Chapter 6, Display and Error Messages.

If you drop the meter or think it is not giving accurate results, make sure that the test strips and control solution have not expired, then perform a control test. If the control result is out of the acceptable range, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.


Test the display before each blood glucose test to make sure it is working properly. To test the display, turn off the meter, then press and **hold**  to see the complete display. All the segments should be clear and look exactly like the picture to the right. If one of the segments is missing or looks different from the picture, do not use the meter. Contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Changing the Batteries

1. Use your thumb to slide the battery drawer out of the meter.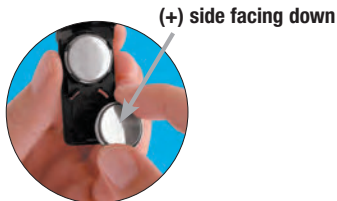

2. Remove the old batteries and place the new ones in the battery drawer with the **(+) side facing down**.

3. Slide the battery drawer back into position until it locks into place.

NOTE

- The meter uses two 3-volt batteries, coin cell type CR2032. This type of battery can be found in many stores. It is a good idea to have spare batteries available.
- Be sure the batteries are inserted **(+) side facing down** or facing away from you.
- Always replace both batteries at the same time and with the same brand.

Display and Error Messages

WARNING

Never make treatment decisions based on an error message.

If you have any concerns, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

(blank display)

The meter will not turn on or the display is blank.


- Batteries are dead. Insert new batteries.
- Display is damaged. Contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
- Meter is defective. Contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
- Extreme temperatures. Move the meter to a more temperate area.


Battery power is low. Change the batteries soon.

The meter is in set-up mode, waiting for you to change or confirm settings.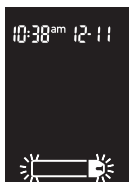


The meter is ready for you to insert a test strip.

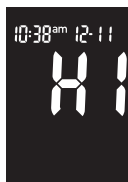Blood glucose may be higher than the measuring range of the system. See Chapter 3, Unusual Test Results.

The meter is ready for a drop of blood or control solution.

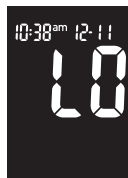Blood glucose may be lower than the measuring range of the system. See Chapter 3, Unusual Test Results.

A pre-meal marker was assigned to this test result.

A general marker was assigned to this test result.

A post-meal marker was assigned to this test result.

Blood glucose is below the defined hypoglycemia (low blood glucose) level. See Chapter 3, Unusual Test Results.

A pre-meal marker was assigned to this test result and the post-meal test reminder has been activated.

The test strip may be damaged. Remove the test strip and reinsert it, or replace it if damaged.

Your blood glucose may be extremely high or a meter or a test strip error has occurred. If your test result matches how you feel, contact your healthcare professional immediately. If your test result does not match how you feel, repeat the test and

refer to Chapter 3, Unusual Test Results. If this test result does not match how you feel, perform a control test with the control solution and a new test strip. If the control result is within the acceptable range, review the proper testing procedure and repeat your blood glucose test with a new test strip. If the E-3 code still appears for your blood glucose test, your blood glucose result may be extremely high and above the system's reading range. **Contact your healthcare professional immediately.** If the control result is not within the acceptable range, see Chapter 2, Understanding Out-of-Range Control Test Results.

Not enough blood or control solution was drawn into the test strip for measurement or was applied after the test had started. Discard the test strip and repeat the test.

An electronic error occurred, or in rare cases, a used test strip was removed and reinserted. Turn the meter off and on, or take the batteries out for 20 seconds and reinsert them. Perform a blood glucose or control test. If the problem persists, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Blood or control solution was applied to the test strip before the flashing drop symbol appeared on the display. Discard the test strip and repeat the test.

The temperature is above or below the proper range for the system. Refer to the test strip package insert for system operating conditions. Move to an area with the appropriate conditions, wait 5 minutes, and repeat the test. Do not artificially heat or cool the meter.

The batteries are almost out of power. Change the batteries now. If the message reappears after the batteries have been replaced, reset the meter. To reset the meter, slide the battery drawer out of the meter, press any meter button, then reinsert the battery drawer back into position.

The time and date settings may be incorrect. Make sure the time and date are correct and adjust, if necessary.

NOTE

If you see any other error display, contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Chapter 7: Technical Information

Product Limitations

Please read the literature packaged with the test strips and control solution for the latest information on product specifications and limitations.

Specifications

Blood volume	Refer to the test strip package insert.
Sample type	Refer to the test strip package insert.
Measuring time	Refer to the test strip package insert.
Measuring range	Refer to the test strip package insert.
Test strip storage conditions	Refer to the test strip package insert.
Meter storage conditions	Temperature: -13–158 °F
System operating conditions	Refer to the test strip package insert.
Relative humidity operating range	Refer to the test strip package insert.
Memory capacity	500 blood glucose results and 20 control results with time and date
Automatic power off	2 minutes
Power supply	Two 3-volt lithium batteries (coin cell type CR2032)
Display	LCD
Dimensions	69 × 43 × 20 mm (LWH)
Weight	Approx. 40 g (with batteries)
Construction	Hand-held

Protection class	III
Meter type	The ACCU-CHEK Nano meter is suitable for continuous operation.
Control solution storage conditions	Refer to the control solution package insert.

Electromagnetic Compatibility

This meter meets the electromagnetic immunity requirements as per ISO 15197 Annex A. The chosen basis for electrostatic discharge immunity testing was basic standard IEC 61000-4-2. In addition, it meets the electromagnetic emissions requirements as per EN 61326. Its electromagnetic emission is thus low. Interference from the meter to other electrically-driven equipment is not anticipated.

Performance Analysis

Refer to the test strip package insert.

Test Principle

Refer to the test strip package insert.

Product Safety Information

WARNING

- **Choking hazard. Small parts. Keep away from children under the age of 3 years.**
- **Strong electromagnetic fields may interfere with the proper operation of the meter. Do not use this meter close to sources of strong electromagnetic radiation.**
- **To avoid electrostatic discharge, do not use the meter in a very dry environment, especially one in which synthetic materials are present.**

Travel Documentation

If you are traveling on a commercial airline, you may be required to provide documentation certifying that this meter meets environmental conditions and test procedures for Airborne Equipment (RTCA/DO-160F) section 21 Emission of Radio Frequency Energy. Please visit accu-chek.com or contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072 to obtain a copy of the document.

Disposing of the Meter, Test Strips, Lancets, and Batteries

WARNING

- Any product coming in contact with blood is considered contaminated (potentially infectious).*
- During normal testing, any blood glucose meter may come in contact with blood.
- Lancing devices may also be considered sharps. Disposal of sharps is regulated by law in many jurisdictions.

The European Union has a requirement for improving waste management practices for certain electronic equipment, but meters fall outside the scope of the European Directive 2002/96/EC.** This is not a requirement for the United States; however, Roche is committed to recycling and sustainability. Comply with any laws or ordinances relating to the disposal of sharps and/or contaminated products. Contact your local health department or other appropriate authorities for proper handling and disposal of used meters, used test strips, used lancets, and used batteries. Please consider the following points when disposing of used testing materials:

- Consider recycling of the meters and batteries at an appropriate facility. Be aware the meter is potentially hazardous electronics scrap (e-scrap) and should be disposed of accordingly. The batteries are potentially hazardous also and should be disposed of accordingly.
- Disinfect the meter before recycling or disposing.

*29 CFR 1910.1030 – Bloodborne pathogens

**Directive 2002/96/EC – Directive on waste electrical and electronic equipment (WEEE)

Explanation of Symbols

CAUTION! Refer to safety-related notes in the instructions for use accompanying this product.

3-volt batteries coin cell type CR2032

Warranty

ACCU-CHEK Nano Meter 30-day Money-back Guarantee for Qualifying Consumers

Roche Diagnostics offers qualifying consumers that purchase an ACCU-CHEK Nano meter, a 30-day money back guarantee. If you are not fully satisfied with your ACCU-CHEK Nano meter, contact the ACCU-CHEK Customer Care Service Center toll-free at 1-800-858-8072 to determine whether you qualify to receive a full refund within 30 days of purchase. If you are covered under Medicare, Medicaid, other federal/state programs, or private insurance you are NOT eligible for this money-back offer. Consumers affected by this exclusion may instead request a different ACCU-CHEK meter/system. The refund will be limited to the amount paid by you net of any rebates. You must have a copy of the dated itemized purchase receipt and the original packaging to obtain this refund.

ACCU-CHEK Nano Meter Limited 3-Year Warranty

Roche Diagnostics warrants to the original purchaser of the meter that your ACCU-CHEK Nano meter will be free from defects in materials and workmanship for three years from the date of purchase. If, during this 3-year period, the meter does not work properly because of a defect in materials or workmanship, Roche Diagnostics will replace it with a new ACCU-CHEK Nano meter or equivalent product free of charge. The warranty on the replacement meter will expire on the date of the original warranty expiration or 90 days after the shipment of a replacement system, whichever period is longer. The purchaser's exclusive remedy with respect to the ACCU-CHEK Nano meter shall be replacement.

This warranty does not apply to the performance of an ACCU-CHEK Nano meter that has been damaged by accident or has been altered, misused, tampered with, or abused in any way. Roche Diagnostics will handle meters that show damage or abuse according to its Non-Warranty Service Policy described on the following page.

THE ABOVE WARRANTY IS EXCLUSIVE OF ALL OTHER WARRANTIES, AND ROCHE DIAGNOSTICS MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL, OR PUNITIVE DAMAGES ARISING FROM OR IN ANY WAY CONNECTED WITH THE PURCHASE OR OPERATION OF THE METER OR ITS PARTS. NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IF ANY IS IMPLIED FROM THE SALE OF THE METER, SHALL EXTEND FOR A LONGER DURATION THAN THREE YEARS FROM THE DATE OF PURCHASE.

Some states do not allow limitations on how long an implied warranty will last or the exclusion of incidental or consequential damages, so the above limitation and exclusion may not apply to you. This warranty gives you specific legal rights, which vary from state to state.

Non-Warranty Service Policy

Roche Diagnostics Non-Warranty Service Policy applies to meters where the above warranty has not become effective, has become inapplicable, or has expired. Roche Diagnostics will replace, at its option, meters returned to it for a service charge (not to exceed \$35). Replacement will be with the same or similar product. Replacement meters will be warranted for a period of 90 days from shipment under a limited warranty providing for replacement of parts and labor at no charge.

Warranty and Service Instructions

All requests for return of ACCU-CHEK Nano meters under the above warranty or service policy must be made to the ACCU-CHEK Customer Care Service Center. You will be mailed a return authorization label, which must be affixed to your carton for shipping the system to Roche Diagnostics. Cartons received without this label will be returned to you at your expense.

Customers experiencing difficulties should review the troubleshooting information in Chapter 6 of this booklet. Further inquiries should be directed to the ACCU-CHEK Customer Care Service Center.

Be sure to fill out and mail the Warranty Card that comes with the ACCU-CHEK Nano SmartView system.

Limited License

WARNING

CAUTION – A RESTRICTED LICENSE LIMITS USE OF THE ACCU-CHEK NANO SMARTVIEW SYSTEM – READ CAREFULLY THE LIMITATIONS RECITED BELOW.

The ACCU-CHEK Nano SmartView system (meter and test strips) and its use are protected by U.S. Patent Nos. 6,645,368 (expires 22-December-2017); 5,352,351 (expires 8-June-2013); 7,276,146 (expires 4-October-2022); 7,276,147 (expires 4-October-2022); 7,407,811 (expires 9-May-2020); 7,452,457 (expires 2-May-2026); 7,488,601 (1-February-2026); 7,494,816 (29-December-2019); 7,569,126 (expires 28-December-2026); and 7,604,721 (expires 12-August-2026). A license to use the ACCU-CHEK Nano SmartView system is required until the expiration of the last-to-expire patent listed above and is only granted when the ACCU-CHEK Nano meter is used with the ACCU-CHEK SmartView test strips.

ACCU-CHEK SmartView test strips are specifically manufactured for operation with the ACCU-CHEK Nano meter. Use of other test strips supplied by another manufacturer may prevent or impair the proper function of the ACCU-CHEK Nano SmartView system.

Using the ACCU-CHEK Nano SmartView system indicates your acceptance of the restricted license to use the ACCU-CHEK Nano meter only with ACCU-CHEK SmartView test strips. Further, if you have purchased an ACCU-CHEK Nano SmartView blood glucose monitoring kit or an ACCU-CHEK Nano meter that includes this restricted license, then this restricted license applies regardless of any additional offers found in ACCU-CHEK SmartView test strip packages. If you do not agree to the terms and conditions of the restricted license, you may return, at the place of purchase, the unused ACCU-CHEK Nano SmartView system for a full refund. If you have any questions, please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Except where prohibited by statute, all warranties covering the ACCU-CHEK Nano SmartView system are voided by use of the ACCU-CHEK Nano meter with any test strips other than ACCU-CHEK SmartView test strips.

WARNING

A RESTRICTED LICENSE LIMITS USE OF THE ACCU-CHEK FASTCLIX SYSTEM (lancing device and lancet drums). READ CAREFULLY THE LIMITATIONS RECITED BELOW.

The ACCU-CHEK FastClix system (device and lancet drums) and its use are protected by U.S. Patent Nos. Re. 35,803 (expires 9-December-2012); 6,419,661 (expires 3-March-2020); 7,077,828 (expires 3-March-2020); 7,322,998 (expires 3-March-2020); and 7,785,338 (expires 5-January-2026). A license to use the ACCU-CHEK FastClix system is required until the expiration of the last-to-expire patent listed above and is only granted when ACCU-CHEK FastClix lancet drums are used with the ACCU-CHEK FastClix device.

ACCU-CHEK FastClix lancet drums are high precision components that are produced to the close tolerances required for satisfactory operation with the ACCU-CHEK FastClix device. Use of other lancet drums with the ACCU-CHEK FastClix device may prevent or impair proper function of the ACCU-CHEK FastClix device.

Using the ACCU-CHEK FastClix device indicates your acceptance of the restricted license to use the ACCU-CHEK FastClix device only with ACCU-CHEK FastClix lancet drums. Further, if you have purchased an ACCU-CHEK FastClix device that includes this restricted license, then this restricted license applies regardless of any additional offers found in ACCU-CHEK FastClix device packages. If you do not agree to the terms and conditions of the restricted license, you may return, at the place of purchase, the unused ACCU-CHEK FastClix device for a full refund. If you have any questions, please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Except where prohibited by statute, all warranties covering the ACCU-CHEK FastClix device are voided by use of the ACCU-CHEK FastClix device with any lancet drums other than ACCU-CHEK FastClix lancet drums.

Additional Supplies

Test Strips

ACCU-CHEK SmartView 50-ct test strips

ACCU-CHEK SmartView 100-ct test strips

Control Solution

ACCU-CHEK SmartView control solution

Lancets

ACCU-CHEK FastClix 102-ct lancet drums

AST Cap

Available upon request from the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

References

- ¹ FDA Public Health Notification: “Use of Fingertick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication, (2010). Update 11/29/2010.” <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>. Accessed February 2, 2012.
- ² CDC Clinical Reminder: “Use of Fingertick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens, (2010).” <http://www.cdc.gov/injectionsafety/Fingertick-DevicesBGM.html>. Accessed February 2, 2012.
- ³ Healthcare Infection Control Practices Advisory Committee (HICPAC), William A. Rutala, Ph.D., M.P.H., and David J. Weber, M.D., M.P.H. Centers for Disease Control and Prevention, 2008. “Guideline for Disinfection and Sterilization in Healthcare Facilities.” Atlanta.

Index

B

- batteries,
 - changing, 80
 - installing, 80
 - type, 80, 87, 91
- beeper, setting, 55
- blood glucose, testing, 25
- button, power/set, 13–14

C

- cleaning,
 - lancing device, 67–68, 74–77
 - meter, 67–73
- comfort dial, 14, 28
- computer, connecting your meter to, 65
- control solution, 18
- control test, performing, 19–22
- control test results, understanding out-of-range, 23–24

D

- dehydration, 10
- disinfecting,
 - lancing device, 67–68, 74–77
 - meter, 67–73
- display check, 79
- display messages, 81–86

E

- error messages, 84–86

F

- frequent urination, 10

G

- general marker, 44–45

H

high blood glucose, 47
hyperglycemia, symptoms, 47
hypo alert, setting 62–64
hypoglycemia, 10, 47
hypoglycemia, symptoms, 47

L

lancet drum, 14, 26
low blood glucose, 47

M

maintenance, meter, 79
memory, meter 49–51

P

patent information, 94–95
post-meal marker, 44–45, 51
post-meal test reminder, 44–45, 52, 57–58
pre-meal marker, 44–45, 51
product limitations, 87
product specifications, 87–88

S

safety information, 9, 89
settings, meter, 52
supplies, 96
symbols, 91
symptoms,
 hyperglycemia, 47
 hypoglycemia, 47

T

technical information, 87–91
test reminders, setting, 59–61
test strips, ACCU-CHEK SmartView, 14–15
time and date, setting, 53–54
troubleshooting, 79–86

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©2012 Roche Diagnostics. 06333630001-0112
ACCU-CHEK, ACCU-CHEK NANO, ACCU-CHEK SMARTVIEW, ACCU-CHEK NANO SMARTVIEW, SOFTCLIX, and FASTCLIX are trademarks of Roche.
All other product names and trademarks are the property of their respective owners.
Assembled and distributed in the U.S.A. by: