


# Simple 3-Step Testing

# 1

## Step 1: Insert the Test Strip

Turn on the meter by inserting the test strip. No Coding required.


 "Your Prodigy® meter is on. Please apply blood to the test strip."



# 2

## Step 2: Apply Blood

Apply a tiny blood sample when the "●" appears on the display. The meter beeps and then counts down from 6 to 1.


 "Now testing."



# 3

## Step 3: Hear Results

Hear and see your accurate results in 7 seconds.

 "Your blood glucose 6.6 millimoles per liter."



## Recalling Test Results


With the meter turned off, press the "M" button twice. The 7-day average will appear, indicating that you are in the memory mode.

If you continue to press the "M" button, the 7, 14 and 28-day averages will appear. After the averages, you can then review the individual test results (450 maximum) stored in the memory.



## Confirmation Window

With the test strip inserted, bring the meter close to your finger or other test site and place beside the blood drop. Blood is automatically drawn into the absorbent channel once the test strip is close enough to the blood drop. DO NOT smear blood onto the test strip or place the blood on top of the test strip.

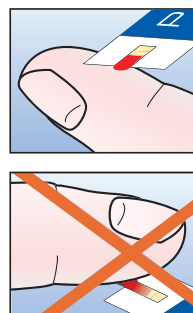

For Customer Care call: 1.800.243.2636  
[www.prodigymeter.com](http://www.prodigymeter.com)

# PRODIGY®

Prodigy Diabetes Care, LLC  
P.O. Box 481928  
Charlotte, North Carolina 28269 USA

©2012 Prodigy Diabetes Care, LLC. Prodigy® and Prodigy AutoCode® are registered trademarks of Prodigy Diabetes Care, LLC. All rights reserved. AQR16 Rev. 4 07/12

# PRODIGY

**AutoCode®**  
Blood Glucose Monitoring System



## Quick Reference Guide